

How to OPERATE THE IONZYME® DF MACHINE

Start the treatment with the Preparation Phase, then continue with the below steps.



Using the Sonophoresis Probe(s):

Using Field Iontophoresis:




Before starting the Sonophoresis or Iontophoresis treatment, ensure that the preferred settings are preselected.

Select Sonophoresis treatment program 1, 2 or 3



Select Field Iontophoresis treatment program 4 or 5

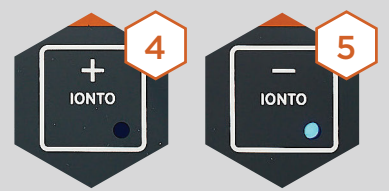

For Dual Technology, ensure that you have placed the passive pole under the arm, in the hand or under the lower back as shown. Not required for program 3.



For Field Iontophoresis, ensure that you have placed the passive pole under the arm, in the hand or under the lower back as shown.



Apply selected professional serum, then continue with the below steps.



Press the silver dial to start the treatment. The timer will start counting down. Once contact is made with the skin, the LED indicator lights will indicate the actual current flow. Use the silver dial to adjust the Iontophoresis mA output.



Once the machine has been set to the desired Iontophoresis level, switch the Sonophoresis on by pressing the button on the probe.



Once the machine has been set to the desired Iontophoresis level, let the treatment continue for 20 minutes.

At the end of the treatment, the timer will automatically count down to 0 seconds. At this point, the device will make 5 short beeps and 'OFF' will flash on the panel.



Conclude the treatment and discuss professional treatment plan and homecare recommendation with your client.

#FacialLikeNoOther
#EnvironSkinCare